







COVID-19 Eye Protection Guidance: Schools and Child Care Centres

When should staff wear eye protection?

- In Child Care Centres eye protection and a medical mask must be worn at all times.
- In schools eye protection should be worn in addition to a medical mask when a physical distance of 2 meters can not be consistently maintained.

Options	Considerations in Selection
Goggles	 Provide the most reliable eye protection from splashes, sprays and respiratory droplets with the snug fit around the eyes. Comes with an adjustable band to allow a firmly secured fit.
Face Shield	 Must cover the front and sides of the face and extend to the chin to reduce the possibility of splash, spray or respiratory droplets from going around the edges of the shield. Does NOT filter respiratory droplets – droplets can escape around the sides of the face shield. Does NOT replace the need for a medical mask.
Safety Glasses	 Does NOT provide the same level of protection from splashes, sprays and respiratory droplets as goggles or face shields. Ensure there is a snug fit with no gaps between the glasses and your face. This is to provide a barrier to prevent splashes/sprays from the front, the sides and the top of the eyes when your head is lowered. OT replace the need for eye protection; they may be worn underneath face

asses do not replace the need for eye protection; they m shields and under some types of protective eye wear.

Additional Tips:

- Do NOT share your eye protection with others.
 Staff should have their own and label it.
- Make sure the type of eye protection worn is suited to the work task(s) assigned. Ensure proper fit of eye protection, and that it does not interfere with the fit of your mask.
- Ensure eye protection will not slip or require frequent handling and adjustment.
- Discard eye protection if it is damaged (e.g., scratched, pitted, broken, bent, ill-fitting, interferes with vision, no longer fastens securely).

Cleaning eye protection:

All eye protection should be cleaned and disinfected between each use. When manufacturer instructions for cleaning and disinfection are not available, follow these steps:

- 1. Clean hands.
- 2. Wipe the inside (the less soiled part), followed by the outside (the more soiled part) of the eye protection using a disinfecting wipe or disinfecting solution.
- 3. Ensure all surfaces remain wet for the disinfectant contact time (e.g. 1-3 minutes).
- 4. Rinse with tap water and allow to dry on a clean surface such as on a clean paper towel.
- 5. Clean hands.
- 6. When dry, store in a clean, dry container labelled with your name and store in a designated area.

Putting on and taking off a medical mask and eye protection

Putting on mask and eye protection	Removing mask and eye protection
Clean hands (for 15 seconds)	Clean hands (for 15 seconds).
 Position mask with the nosepiece on top and place the loops of the mask over each ear. 	Remove eye protection.
Stretch the mask to fit snugly under the chin. Bend the nose bar over the bridge of the nose so that the mask fits snugly over the nose and the mouth.	 Clean and disinfect the eye protection (see below). Remove mask by pulling the loops of the mask forward and away from the face. Avoid touching the outside. Discard into the garbage.
Put on eye protection.	Clean hands.

References:

Putting on mask and eye protection, Public Health Ontario, video: https://www.publichealthontario.ca/en/videos/ipac-maskeyes-on

Taking off mask and eye protection, Public Health Ontario, video: https://www.publichealthontario.ca/en/videos/ipac-maskeyes-off

Cleaning and disinfection of reusable eye protection, Public Health Ontario: https://www.publichealthontario.ca/-/media/documents/ncov/ipac/2021/03/covid-19-infographic-cleaning-eye-protection.pdf?la=en

Frequently Asked Questions: Schools and COVID-19, Nov 24, 2020 https://www.publichealthontario.ca/-/media/documents/ncov/sch/2020/12/covid-19-fag-schools.pdf?la=en

Adapted from Ottawa Public Health with permissions from COMOH School Working Group







